

Gebrauchsanweisung
User manual
Modo de empleo

MANO

DE

EN

ES



Before initial use

Insert the rechargeable battery into the case


- Take MANO out of the plastic bag
- Open the folding stand until it locks in place
- Slide open the battery cover pushing in the direction of the arrow 1
- Remove the battery cover
- Take the rechargeable battery out of the packaging
- Insert the rechargeable battery into the case, making sure the contacts are properly positioned
- According to the illustration, slide the battery cover plate back into the case (Make sure that the holes on the battery cover fit squarely over the pins on the case)
- Press the battery cover plate against the battery. Push in the direction of the arrow 2 until it locks in place

Quick reference guide

- Push the **yellow button**: switch MANO on/off
- Push the **zoom button**: change the color modes
- Push the **zoom button** upwards/downwards: zoom in/out
- Push the **snapshot button** in live image: save an image
- Press the **snapshot button** upwards:
switch from freeze frame to live image
- Press the **snapshot button** downwards:
the next image will be shown
- Keep **snapshot** pressed: scroll zoomed image
(Version 2.0 or higher)

Skizzen:/Illustrations:/Ilustraciones:!

Vor Inbetriebnahme/Before initial use/ Antes de usar

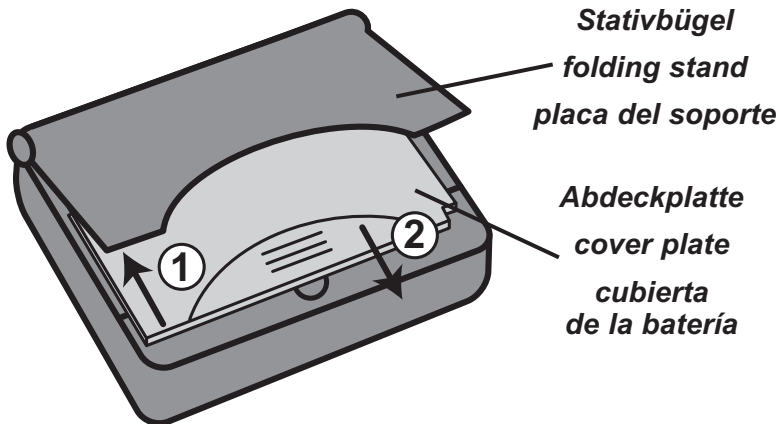

Kurzanleitung/Quick reference guide/ Guía de referencia rápida



Schnappschuss
snapshot
foto

Ein/Aus
on/off
Encender/apagar

Zoom
zoom
Zoom/enfoque



Content

Accessories	28
Notes before operating the device	28
1. Start-up	29
2. Description of the controls	29
2.1 Yellow “on/off” button	30
2.2 “Zoom” Button with 4 functions	30
2.2.1 Zoom plus = Increase Magnification	30
2.2.2 Zoom minus = Decrease Magnification	30
2.2.3 Changing color modes	30
2.2.4 Turn on/off the light	30
2.3. Snapshot/Live image Button	31
2.3.1 Saving an image	31
2.3.2. Changing to Live picture	31
2.3.3. Retrieving saved pictures	31
2.3.4 Widescreen-function	31
3. Application	32
3.1 Reading with stand	32
3.2. Handheld reading	33
3.3 Writing	33
4. Battery operation	34
4.1 Recharging the battery	34
4.2 Rechargeable battery display	34
4.3 Display of rechargeable battery flashes	34

5. Power Supply	35
5.1. Included adapters	35
5.2. Connecting the adapter to the power supply	35
5.3 Connecting MANO to the power supply	36
6. Standby function	36
7. Automatic switch-off	36
8. Saving the settings	36
9. Notes for changing the rechargeable battery	36
9.1. Changing of the rechargeable battery	37
9.2. Non-liability in case of using wrong rechargeable battery	38
9.3 Disposal of the rechargeable battery	38
9.4 Protective circuit of the rechargeable batteries	38
10. Protective bag	39
11. Hand strap	39
12. Notes for handling	40
13. Cleaning of the device	41
14. CE and FCC notes for electro-magnetic compatibility (EMV)	41
15. Technical data	42
16. Warranty Information	42

MANO is an electronic magnifier that magnifies text and images.

You can use it at home or take it with you when you go shopping or on a visit.

Accessories

The following equipment is included in the packaging:

- a power supply with four country-specific adapters (country-specific top parts guarantee a world-wide application)
- a hand strap
- a protective bag
- a manual in three languages:
German, English, Spanish
- a microfibre tissue for cleaning the display

Notes before operating the device

!Please note!

Read the manual carefully before switching on the device. Keep the manual in a safe spot for further reference.

If you need to return your MANO for service, pack it in its original packaging and return it to your retailer, indicating the problem you are having.


The manufacturer is not liable for malfunctions or damages caused by inappropriate usage.

1. Start-up

Please insert the pre-charged battery as described in the quick start guide on the front cover page.

MANO can be switched on by pressing the yellow „on/off“ button.

2. Description of the controls

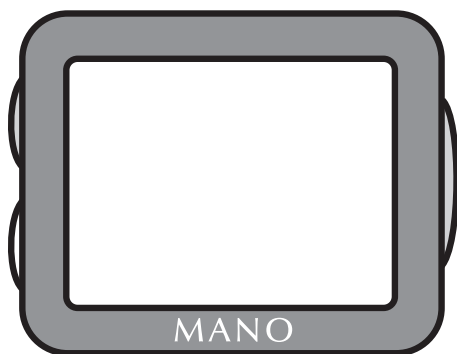

left side



live image
snapshot
image selection

on/off

right side



zoom plus

color mode

zoom minus

Plug
for AC adapter

2.1 Yellow “on/off” button

The on/off button is unmistakably marked in yellow. When you briefly push the button, MANO will be switched on. To turn off MANO, push the yellow button for 2 seconds.

2.2 “Zoom” Button with 4 functions

2.2.1 Zoom plus = Increase Magnification

By pushing the button upwards, you will increase the magnification level

2.2.2 Zoom minus = Decrease Magnification

By pushing the button downwards, you will decrease the magnification level

2.2.3 Changing color modes

By pushing the zoom button you may change the color selection options on the TFT screen:

- Full color
- Semi-colors
- Positive mode: black-white
- Negative mode:
white-black, green-black, yellow-black, yellow-blue

2.2.4 Turn on/off the light

If you wish to turn off the light (for use with ATMs and other machines which may cause glare), push the zoom button for 2 seconds. To turn on the light, push the button briefly once more.

2.3 Snapshot/Live image Button

You can change from freeze frame image to live image with the “snapshot” button. Both text and images can be saved and retrieved.

2.3.1 Saving an image

By pushing the snapshot button, you may save the image currently on the TFT screen.

!Please note! You may save up to 3 pictures.

If 3 pictures are saved and you take another one, the first one will be deleted

2.3.2 Changing to Live picture

By pushing the snapshot button upwards, you will switch back to live viewing.

2.3.3 Retrieving saved pictures

By pushing the snapshot button downwards, you will retrieve the saved pictures. Pushing the button multiple times will advance to subsequent images.

2.3.4 Widescreen-function (Version 2.0 or higher)

This function enables you to read saved images which cannot be shown completely on the display after having been enlarged. This may be used for telephone numbers, email addresses, schedules, package inserts etc.

Proceeding:


Choose a saved image. In case you cannot read the font use the button zoom plus for enlargement. Should the text not be legible due to this operation, permanently press the snapshot button. Move MANO across any reading material to the direction of your choice (to the left, right, up and down). Now the text will be shown on the display according to your movement.

3. Application

MANO may be used in 3 different ways:

3.1 Reading with stand


Expand the folding stand until it locks in place
Place MANO on the reading material. Move MANO with the expanded folding stand across the reading material. Now you can see your reading material or your pictures on the TFT screen.

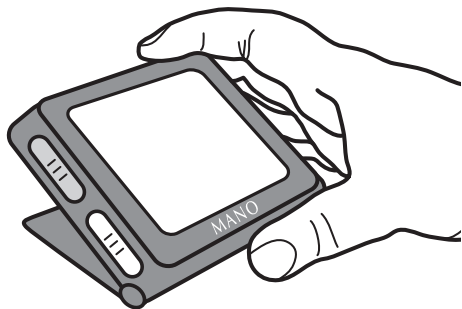

The angle of the TFT screen is about 20°. This enables an ergonomic seating position and an optimal depiction of contrast and brightness.

3.2 Handheld reading

Fold down the stand frame and hold MANO at a distance of 1 to 15 cm (0.5" to 6") above the reading material. Within that range, a sharp image can be displayed on the TFT screen.

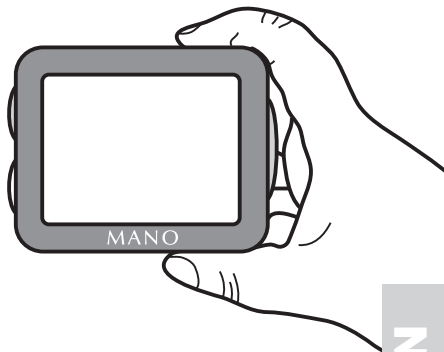


!Please note! Viewing of computer monitors

If you wish to use MANO to view screens such as ATM machines, computers, or other electronic displays, you may experience excessive glare due to the internal lighting on MANO. To avoid the glare, turn off the internal light on MANO (see 2.2.4).

3.3 Writing

If you lift MANO or open the folding stand, you may write short notes.

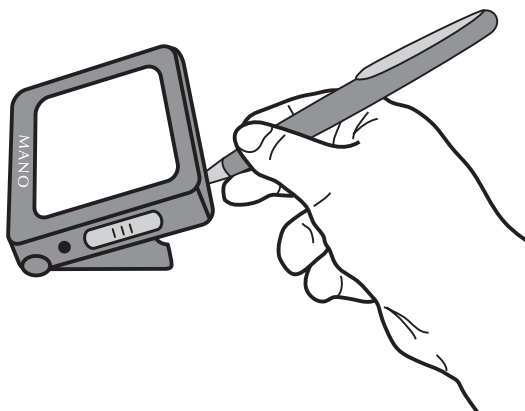

4. Battery operation

4.1 Recharging the battery

Switch off MANO

Put DC plug of power supply in DC nut of MANO

Plug the small end of the power supply into the power port on the MANO.

Plug the power supply into an electrical outlet.

The rechargeable battery will start charging.

Charging time is approximately 2.5 hours.

4.2 Rechargeable battery display

Upon turning MANO on, you will see the symbol of the rechargeable battery on the TFT display. The charge level will be indicated.

Charge level 100% = rechargeable battery is completely charged

Charge level 50%=rechargeable battery is half charged

If the symbol of the rechargeable battery is flashing, the rechargeable battery needs to be charged

4.3 Display of rechargeable battery flashes

If the rechargeable battery icon flashes, the rechargeable battery needs to be charged immediately.

If you do not begin charging promptly, MANO will shut itself down.

5. Power Supply

Included in the delivery are a 100-240 VAC power supply and 4 primary adapters. They will enable you to connect the power supply world-wide.

5.1 Included adapters

1 adapter for Europe (EU)


1 adapter for England (UK)

1 adapter for USA, South America, Japan etc. (US/J)

1 adapter for Australia (AUS)

5.2 Connecting the adapter to the power supply

When connecting the primary adapter, the power supply must not be plugged into an electrical outlet. Choose the primary adapter necessary for your region and connect it to the power supply. Push both parts together until they lock in place.

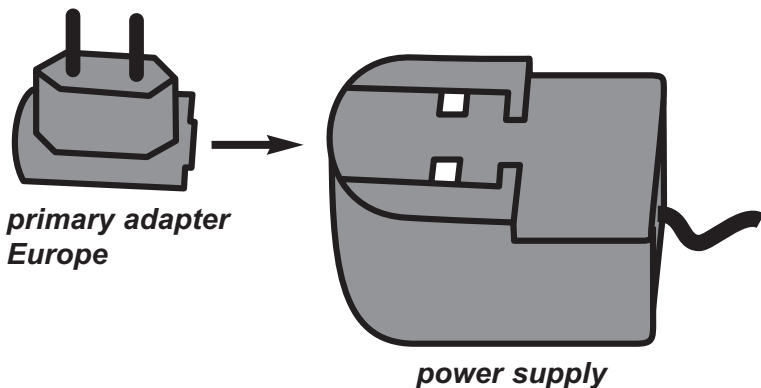

5.3 Connecting MANO to the power supply

After connecting the primary adapter, plug the power supply into an electrical outlet. Switch on MANO. The operating elements and the use are the same as for battery operation. If you use MANO while plugged into an electrical supply, the power supply to the battery is disconnected in order to preserve battery capacity.

6. Standby function

If MANO is left idle for 2 minutes of no use, the device switches automatically to standby mode. Simply moving the device turns it on again.

7. Automatic switch-off

If MANO remains in the standby mode for more than 5 minutes, the unit will shut off completely. MANO will then have to be switched on by means of the main on/off button.

8. Saving the settings


Upon turning off the device, MANO will save all data and settings. They will be activated as soon as you switch the device back on.

9. Notes for changing the rechargeable battery

You may only use the rechargeable battery specified for MANO. A replacement can only be purchased from Reinecker Reha-Technik GmbH or from one of its contractual partners.

9.1 Changing of the rechargeable battery

Switch off MANO and pull the DC plug of the power supply out of the DC nut of MANO. Open the folding stand. Push the cover plate of the rechargeable battery softly out of its lock (1), take out the cover plate (2). Take the rechargeable battery out of the bay (deepening of the handle at the edge). Put the original rechargeable battery into the bay (Beware of the positioning due to the contacts). Put the cover plate with the fasteners in front of the lock holders onto the rechargeable battery. Press on the plate evenly and into the direction of the LEDs. Push all the way until stop.

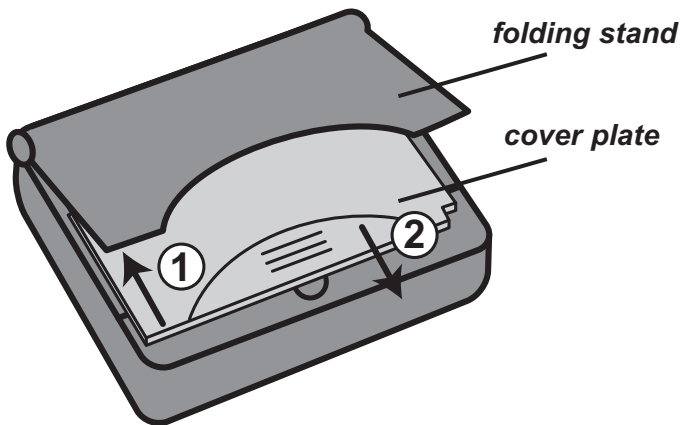

9.2 Non-liability in case of using wrong rechargeable battery

!Please note!

You may only use the rechargeable battery specified for MANO.

The usage of rechargeable batteries other than one provide by Reinecker Reha-Technik GmbH or one of its partners is specifically prohibited, as this may cause severe damage to the device.

The MANO warranty is voided if any other rechargeable battery is used.

Reinecker Reha-Technik is not liable for damage caused by wrong or faulty rechargeable batteries as well as inappropriate operation when using the rechargeable battery, be it damage to property or persons.

9.3 Disposal of the rechargeable battery

Rechargeable batteries must be disposed of according to the laws of the specific countries. They must not be burned. Do not allow animals or children to play with the battery.


9.4 Protective circuit of the rechargeable batteries

Despite the protective circuit applied at the rechargeable battery, it is prohibited to short-circuit the contacts of the rechargeable battery with a metallic item.

This may cause the rechargeable battery to explode, creating potential danger for anyone nearby.

10. Protective bag

Please always use the protective bag for storing MANO. The display and the case will thus be protected from damage.

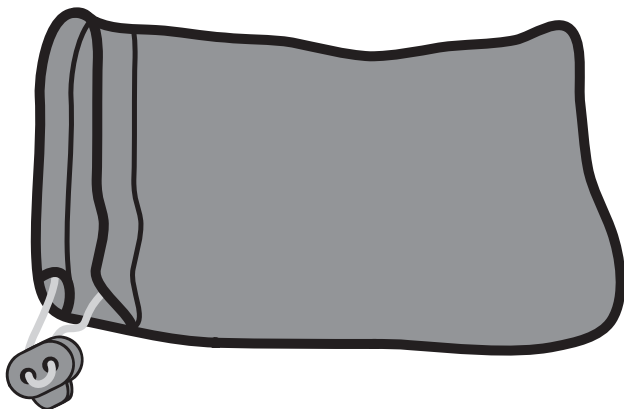

11. Hand strap

In the cardboard box a hand strap is included. It may be attached to carry MANO on your wrist.



12. Notes for handling

!Please note!

- The pictures used in this manual only serve illustrative purposes and may not be in accordance with the actual positions. Content changes of this manual are subject to change without advance notice.
- Do not expose MANO to extreme direct sunlight. For instance, do not store it close to windows or heaters.
- Protect MANO from excessive heat and do not place other objects on top of it.
- Keep all liquids away from MANO.
- Use MANO only in dry surroundings.
- Never open the case, as this may only be done by an authorized speciality retailer.
- Operate MANO only with the indicated supply voltage.
- Protect MANO from mechanical damage which may be caused by stroke, blow or fall.
- Do not expose MANO to magnetic fields.
- Do not alter MANO in any way.
- Send MANO only in its original packaging.
- Never use MANO with wet hands.
- Use the fabric bag for carrying in bags or other containers.

13. Cleaning of the device

For cleaning of the display, use the included microfibre tissue or another dry and soft tissue. Do not use any liquids for cleaning. Never clean the display or the case with thinner, gasoline, alcohol, or other chemical means. The case parts may be damaged.

14. CE and FCC notes for electro-magnetic compatibility (EMV)

MANO is suited for use in all facilities, including all homes and offices connected to the public power supply. MANO is destined for operation in the indicated electro-magnetic surrounding. The operator should make sure to use it in a surrounding as described in this manual.

MANO uses HF energy exclusively for its own function. Therefore transient emissions of high-frequency transients are limited. It is unlikely to disturb other devices in operation nearby.

MANO has been verified according to the European regulations of each country as well as to the FCC regulations. It is guaranteed that the limits determined in the regulations will not be exceeded. However, one should consider that common operation of MANO and other devices may lead to interferences between MANO and other devices. For this reason one should not locate the device within proximity of another device. Keep a distance of 1 meter (3 feet.)

15. Technical data

- Magnification: approx. x1 - x20
- Display:
 - Full color
 - Semi-color: positive mode: black on white
negative mode: white on black,
green on black, yellow on black,
yellow on blue
- weight: 139 gr (4.9 oz)
- Measurements of the case: 89 x 74 x 21 mm
(3.5" x 2.9" x 0.8")
- Diameter TFT: 3.5"
 - Active surface: 74 x 54 mm (2.9" x 2.1")
- Electric power supply:
 - 100-240 VAC 50-60HZ secondary 12 VDC 0,5 A
(in combination with a power supplyf)
- Power consumption: approx. 6 watt
- Rechargeable battery: Li-ions 1500 mAh

16. Warranty Information

The legal requirements apply.

Kontakt innerhalb Deutschlands:
Reinecker Reha-Technik GmbH
*Entwicklung, Produktion, Vertrieb,
Kundenberatung und Kundendienst*

Sandwiesenstraße 19
64665 Alsbach-Hähnlein

Tel. 0 18 05-54 56 65*
Fax 0 62 57-25 61

info@reineckerreha.de
www.reineckerreha.de

*Nur gültig innerhalb Deutschlands.
Festnetzpreis 14 ct/min, Mobilfunkpreise abweichend.

International contact:
Reinecker Reha-Technik GmbH
Head office and German engineering

Sandwiesenstraße 19
64665 Alsbach-Hähnlein, Germany

Tel. +49 62 57-93 11 0
Fax +49 62 57-25 61

info@reineckerreha.com
www.reineckerreha.com

Bitte wenden Sie
sich bei Fragen oder
einem Servicefall
direkt an unsere
Zentrale oder Ihren
authorisierten
Fachhändler.

In case service
is necessary,
please contact your
authorized distributor.